

RelyX™ Unicem Self-Adhesive Universal Resin Cement Aplicap™ / Maxicap™

Indicated for Final Cementation of:

- Metal and porcelain-fused-to-metal crowns, bridges, inlays and onlays.
- All-ceramic/composite crowns, bridges, inlays and onlays.
- Strengthened core ceramic crowns and bridges.
- Pre-fabricated and cast posts and pins.

Preparation:

- Remove provisional restoration and all temporary cement residue.
- Trial fit the restoration and clean thoroughly.
- Pretreat restoration according to manufacturer's instructions (e.g. roughen metal and composite surfaces and apply ceramic primer if indicated).


- Clean tooth, rinse and lightly dry.

Do Not Overdry.




Activation:

- Insert capsule into Activator.
- Press down handle **completely and hold for 2-4 seconds.**

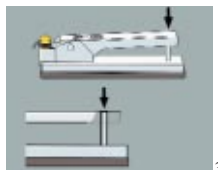

Mixing:

- Insert activated capsule into mixing device (triturator/amalgamator).
- Mix 10 seconds for 3M™ ESPE™ RotoMix™ Capsule Mixing Unit.
- Mix 15 seconds at highest speed for other mixing devices.

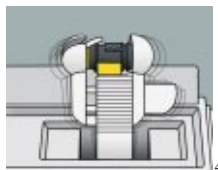

RelyX™ Unicem Self-Adhesive Universal Resin Cement Aplicap™ / Maxicap™



Dispensing:

- Remove capsule from mixing device and insert into Applicator.
- Open nozzle and dispense cement directly onto bonding surface of restoration or directly onto tooth surface for inlays and onlays. For posts and pins, **do not use a Lentulo-Spiral**.
- Working time from start of mix:
 - Aplicap** 2:00 min.
 - Maxicap** 2:30 min.



Placement:

- Seat restoration and hold in place with light pressure.
- Excess cement is best removed in the cement's gel state with scaler or explorer. Tack cure excess with light for 2 seconds or allow to self-cure for 2–3 minutes from start of mix.



Final Cure:

- For translucent ceramic restorations, light-cure each surface for 20 seconds.
- For all others, allow cement to self-cure
 - Aplicap** 5:00 min. from start of mix.
 - Maxicap** 6:00 min. from start of mix.



Finish and Polish:

- Insure all excess cement is removed.
- Adjust occlusion.
- Finish exposed surfaces using appropriate instrument and polishing paste.
- Finish proximal surfaces using 3M™ ESPE™ Sof-Lex™ Discs and Strips.